

Labeling A Custom Board

1. When labeling a custom BxBoard, it is important that each abbreviation clearly represents the site location. When creating abbreviations be sure to not use the same abbreviation for different locations.

Common Abbreviations include:

LES - Lesion

LSV - Left Seminal Vesicle

LTZ - Left Transitional Zone

LAH - Left Anterior Horn

LPM - Left Posterior Medial

LPZ - Left Peripheral Zone

ROI - Region of Interest

RSV - Right Seminal Vesicle

RTZ - Right Transitional Zone



RAH - Right Anterior Horn

RPM - Right Posterior Medial

RPZ - Right Peripheral Zone

IMPORTANT: If a lane is empty, you **MUST** label the lane empty using the Ø symbol.

Example:

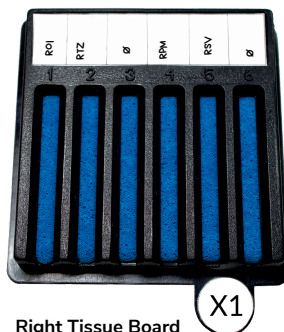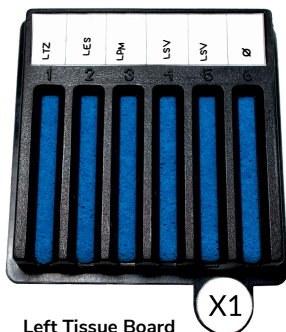

Left and Right tissue MUST be on separate boards.